Product name: ROLLATOR

Item No: 675330

v001: 15/01/2025

# USER MANUAL

## PARTS INCLUDED

- 1 x Rollator Frame
- 2 x Removable Front Wheels
- 1 x Rollator Bag
- 4 x Fasten Knobs

## SPECIFICATIONS

- Overall Width: 25.2"
- Overall Depth: 26.2"
- Armrest Lowest Height: 31.3"
- Armrest Highest Height: 36.6"
- Seat Width: 18"
- Seat Depth: 9.8"
- Folded Width: 10.8"
- Bag Weight Capacity: 11 LBS
- Product Weight: 18 LBS
- Max User Weight: 300 LBS

## WARNING

- Only use the rollator as a walking aid, but do not transport people or objects.
- Ensure the brakes are in the locked. Check the brakes before use.
- Do not hang heavy items from the handles as this may affect the stability.
- Exercise caution when there is a heavy load in the bag (max. 5Kg).
- Do not stand on the rollator or use it as a step.
- Product can not be put outside in rainy conditions.
- Do not use strong corrosive cleaner to clean the surface.

## ASSEMBLY INSTRUCTIONS

A) Remove all packaging. The rollator will be folded as shown in Figure 1

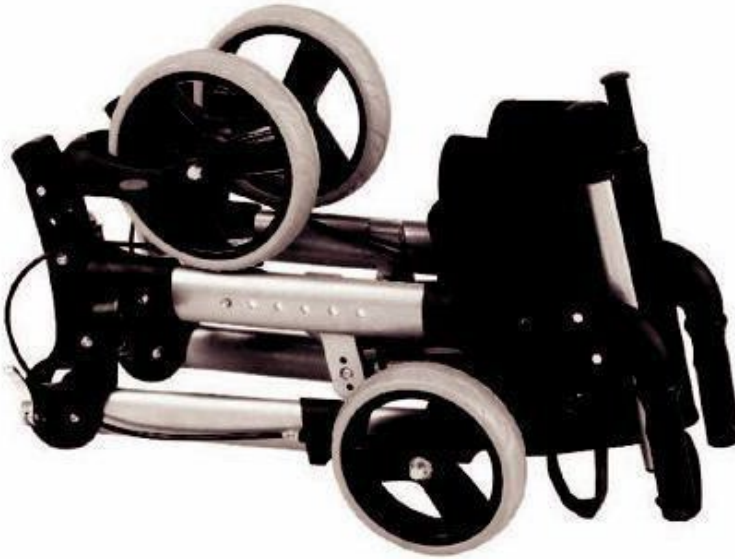
Figure 1

B) Unfold the front wheels and secure in place using the fastening knobs as shown in Figure 2.

Figure 2 Fastening Knobs

C) Unfold the rear legs as shown below and insert knob and turn it to fasten the rollator until stable as shown in Figure 3

Figure 3

D) Open the rollator, push down on either side of the seat as shown in Figure 4. **IMPORTANT:** Both sides must “click” into position.

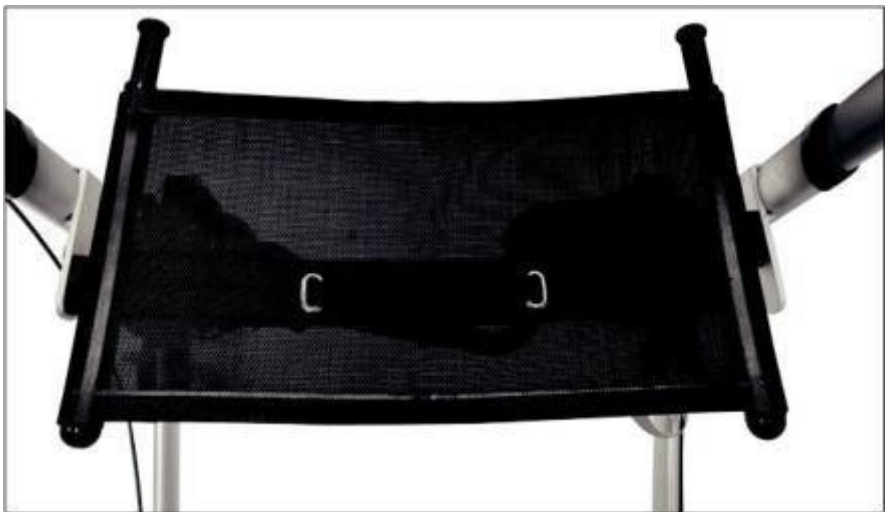
Figure 4

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FOSHAN HCT MEDICAL EQUIPMENT CO., LTD
SUNGO Europe B.V. Fascinatio Boulevard 522,