Product name: Manuel Wheel Chair
Item No: 687970 v001: 30/03/2025

3PAGEN

GB INSTRUCTION MANUAL

Please read and retain these instructions for future reference

Dimensions

Open size: 97 x 56.5 x 42cm/folded: 86 x 31 x 31cm

Seat width: 45cm

- Super lightweight, weighs just 8.9kg which is less than a bag of compost!
- Comes with a fabric suitcase on wheels for easy transportation (suitcase has adjustable handle and folds away small)
- Supplied with 2 in 1 seat pad, use without red seat pad in warmer weather and the mesh is breathable to keep you cool when sitting for long periods
- Fold away fully
- Pocket at the back of the wheelchair
- Height adjustable foot rest
- Brakes on both back wheels
- Arms can lift up and click back into position
- Includes seat belt

Total length	≥1200mm	Total width	≤700mm	Maximum load	≤100kg
Empty vehicle mass	≤25kg	Slope performance	≥8°	Minimum reversing width	≤1500mm
Glide offset	≤350mm	Maximum radius of gyration	≤1400mm	Minimum radius of gyration	≤850mm
Wheel location	All wheels except the lifting wheel must land stably				
Static stability	Longitudinal forward tilt ≥10°, longitudinal backward tilt ≥10°, roll ≥15°				

Packing List

Wheelchair, manual

Main structure

Manual wheelchairs are mainly composed of a frame, cushions, armrests, pedals, front wheels, rear wheels, and brakes.

Scope of application

Manual wheelchair is used as a means of transportation for the disabled, the sick and the elderly and infirm who have mobility difficulties.

Precautions, warnings and informative content

1. It is strictly forbidden for the user to modify the wheelchair. In order to obtain the maximum stability, the axle of the rear wheel must be fixed in the most surface position. Any movement of the wheel axle will affect the stability of the wheelchair during travel. If the axle of the rear wheel of the wheelchair is moved forward, please check whether it is stable before using the wheelchair. The stability of the wheelchair must be tested before the seat cushion of the wheelchair is to be tilted to the limit or the rear wheel axle is moved forward.
In order to ensure safety, it is recommended that users carefully tests whether the wheelchair is stable before using it.
2. Do not use both hands to apply pressure to both sides of the frame or handle to fold. Inappropriate folding or opening of the wheelchair can easily cause damage.
3. The brakes on both sides of the wheelchair are only used for braking when it is parked. **They are not a braking device.** They should not be used for slowing down and braking. If you are unaccompanied, you can hold the hand wheel and brake gradually.
4. Do not use the wheelchair to rush up steps, otherwise it will cause an accident or damage the wheelchair.
5. Do not go up and down steps or stairs without help. When going up and down the stairs, only persons familiar with the use of the wheelchair are allowed to move the occupant together with the wheelchair, otherwise it will cause danger to the occupant and damage the wheelchair.
6. Unless there are special instructions, all wheelchair adjustment operations should be carried out when the wheelchair is not in use.
7. When you are in a wheelchair and want to pick up items other than the wheelchair (such as the front, sides, or back of the wheelchair), please be careful not to lean too much, so as to prevent the wheelchair from losing weight and causing it to fall over.
8. Before using the wheelchair, check whether the armrests, handles, fastening screws, etc. are loose. If so, please contact a professional technician to correct it before normal use. Before using the wheelchair, please practise using the wheelchair with an accompanying person.
9. It is strictly forbidden for the user to have one or two front wheels off the ground, so as not to cause danger and hurt themselves or others. Do not bend down below your knees in order to pick up items on the floor whilst sitting in the wheelchair.
10. Please do not change your sitting posture when you lean back, otherwise the wheelchair will overturn.
11. The wheelchair should not be taken on an escalator, otherwise it will cause serious personal injury. Wheelchair users are strictly forbidden to be transported by any form of transportation vehicles in a wheelchair.
12. When a person in a wheelchair wants to transfer from a wheelchair to a car, first park the car in the nearest and best position, and turn the front wheel of the wheelchair to the place where the transfer is needed. Before getting on the wheelchair, please confirm whether the wheelchair is in the braking state.
13. When carrying the wheelchair, please grasp the fixed connection part of the wheelchair, rather than grasp the movable parts, such as armrests, footrests, etc.
14. Please do not step on the pedal to get on and off the car, be sure to fold the pedal first or roll it to one side. The user must not stand on the pedals during use to prevent the wheelchair from tipping over.
15. When using a wheelchair on a sloping road, do not overturn the wheelchair or change direction suddenly. Do not brake suddenly when going downhill to prevent the danger of falling forward.
16. The rated load of the wheelchair is 100kg.
17. Wheelchair driving is suitable for driving on a flat ground, and not suitable for driving on uneven roads. When encountering obstacles ahead, you should avoid driving in a detour to avoid the danger of the wheelchair tilting. When the user drives the wheelchair by himself, please keep a constant speed, and the driving speed should be kept at 3.5km/h.
18. The slope within 5° is the safe driving range of the wheelchair, for steeper slopes assistance is required from others. When going up and down, keep the corresponding forward and backward positions, and keep the backrest in a vertical position on the slope. Long-term users should wear protective gloves to avoid hand injuries.
19. Do not put your fingers into the wheel to avoid pinching your fingers.
20. The normal service life of the product is 5 years. The product should be disposed of in accordance with local laws and regulations at the end of its life.

Instructions or icons

1. How to open/fold the wheelchair:

When opening the wheelchair, you can press both sides or one side of the seat, and the wheelchair can be opened easily; when folding the wheelchair, please flip up the armrest first, and then lift it up with the seat cushion. The wheelchair can be folded.

2. Foot pedal height adjustment:

Please press down the marble on the foot tube to move up and down, and then adjust the position of the foot pedal.

3. How to use the brake:

There are brakes on both sides of the seat. When parking, be sure to use the brake to park the wheelchair.

4. How to use the stairs:

There is a backward tilting bar with a plastic sleeve at the bottom of the wheelchair. The person pushing the chair can step on the backward tilting bar to make the wheelchair tilt backward. The front wheel first climbs the steps, and then the wheelchair can easily be lifted. Place the wheelchair with the passengers up the steps.

Special storage conditions and methods

The wheelchair folds up to fit into the suitcase bag.

The packed wheelchair should be stored in a temperature not exceeding -10°C -40°C, Humidity should not exceed 80%. It should not be stored near corrosive gas in a well-ventilated and clean room. Keep away from moisture, sunlight, and heat sources during transportation.

Cleaning and Maintenance

- Seat cushion can be gently cleaned with cold water. Do not soak.

DISPOSAL

- We use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.
- Plastics, polybags – Contains the following recyclable plastic.

Code & Symbol	 PVC	 LDPE	 PP	 OTHER
Type of Plastic	Polyvinyl Chloride	Low Density Polyethylene	Polypropylene	PA

- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

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